UM-KINGPWSL-KD Rev. 2 as of 2022/05

MADE IN ITALY

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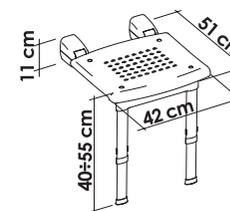
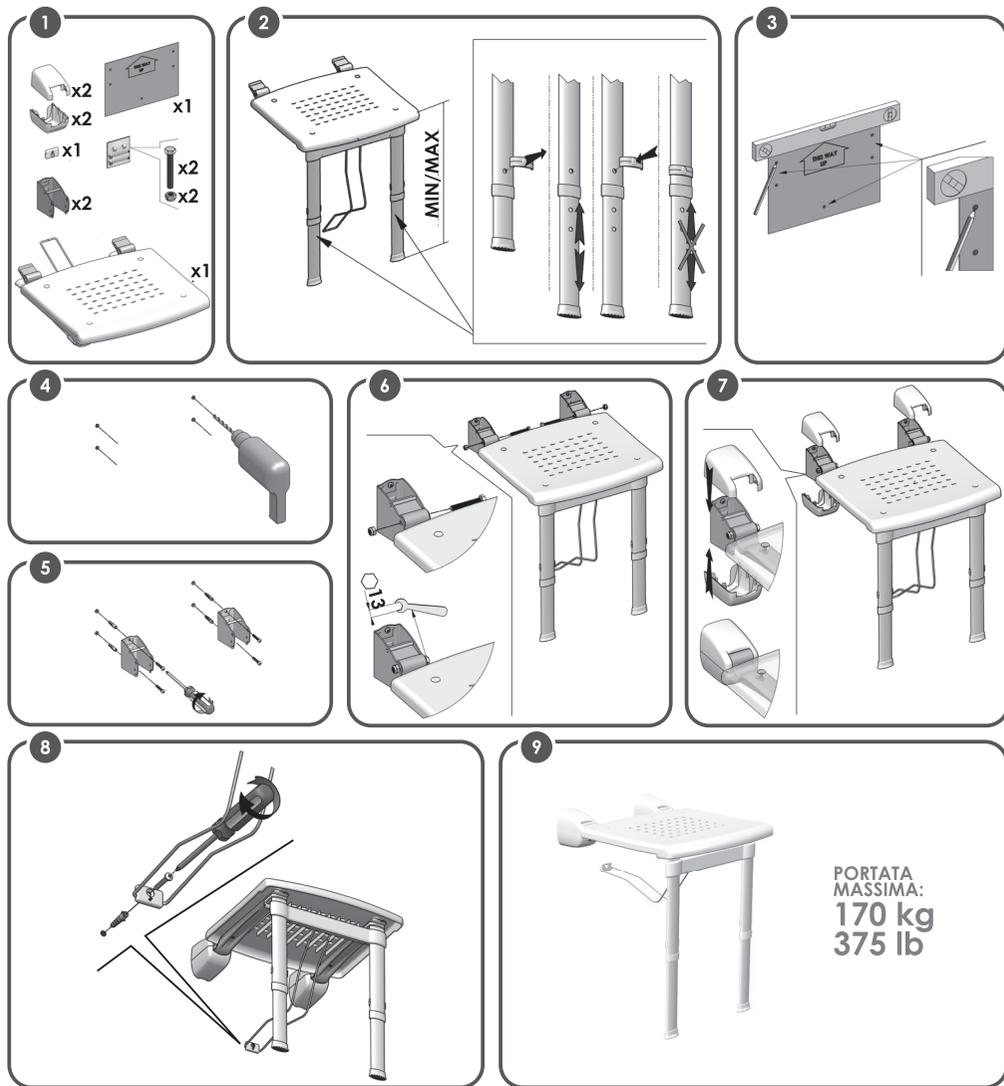
MADE IN ITALY

INTENDED USAGE:

The *Lift-up wall mounted seat with legs* is meant to be used while showering, allowing the user to achieve greater independence, reducing muscle fatigue due to standing position as well as slip and injuries risks. The folding mechanism make these products suitable even for small bathrooms and help the user to keep the shower tray tidy and free from any obstacle without effort.

INDICATIONS AND CONTRAINDICATIONS:

The lift-up wall mounted seat with legs is aimed to elderly people, wheelchair users, people with reduced mobility of lumbar, knee or hip, and to all those who have difficulty to maintain the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during the sitting and standing procedures as well as to have adequate ability to keep sitting. The lift-up wall mounted seat must not be used if the user has problems related to balance or perception disorders.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceed.
- Before every use, be sure the lift-up wall mounted seat is correctly assembled.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not modify the lift-up wall mounted seat and its components in any way.
- Do not use the lift-up wall mounted seat if damaged or altered in any way. Cracks and deformations may strongly affect the performances and safety of the device.
- After use, the lift-up wall mounted seat should be folded in its non-use configuration.
- Do not use the lift-up wall mounted seat for a different purpose from the intended one.
- The lift-up wall mounted seat must not be stored outdoors.

MAINTENANCE INFO:

The products maintenance consists of the cleaning and sanitation procedures. In order to guarantee a safe use and adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before to proceed with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reuse by others, the product must be correctly cleaned and sanitized.

Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the normal household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose it.

INCIDENT REPORTING:

If there was a serious incident involving this device, contact the manufacturer of this product at info@kdesign-group.com and the responsible authority in your country.